

ATdirect®

Page 1/1  
Docident 010820SF0925

## Shelf-lives of prepared use solutions

### Test conditions:

- Use solution prepared with fully desalinated water or tap-water (8°dH)
- Stored in closed polythene and glass bottles, no contamination by dirt.
- Active ingredients measured at predetermined intervals.
- Criterion: active ingredient content must not fall below 90 percent of its initial level.

The data can be regarded as transferable to higher or lower concentration levels.

Product	Solution [%]	Shelf life	Test report
antifect FD 10	0.5	12 months	Derived from test-report No. 83/328, Dr. B. Löwer v. 23.11.1983
buraton 10 F	0.5	12 months	Report No. 83/328 Dr. B. Löwer 23.11.1983
gevisol	1.0	12 months	Report No. 85-066 Dr. B. Löwer 22.03.1985
gigasept	3.0	9 months	Report No. 83/328 Dr. B. Löwer 23.11.1983
gigasept	10.0	18 months	J. Hantelmann, 24.04.1985
gigasept AF forte	1.0 / 2.0 / 5.0	9 months	D. Kunde, 15.01.2008
gigasept FF (neu)	2.0 / 5.0	9 months	D. Kunde, 17.01.2008
gigasept instru AF	1.5 / 3.0	6 months	M. Mohr, 16.12.2004
gigasept med forte	1.0 / 2.0 / 5.0	9 months	D. Kunde, 15.01.2008
grotanat flüssig	2.0	12 months	Report No. 85-066 Dr. B. Löwer 22.03.1985
lysetol FF	2.0	9 months	Report No. 89/136 M. Mohr 16.05.1989
lysetol V	1.0	9 months	Report AT 122/99 R.Kleinwort , 09.05.2000
perform	0.75	30 hours	Graph by R. Kleinwort, 13.02.1986
terralin protect	0.50	12 months	M. Mohr, 03.01.2006

When the containers are being refilled with the solution, they must first be submitted to special hygienic treatment and supervision.

Dr. B. Löwer  
12.11.1997  
rev.: 16.01.2009 (S. Fehr)