



Neutralizer for the automated reprocessing of dental devices

thermodent® neutralizer

Our Plus

- based on citric acid
- prevents deposits and coverings
- high level of material compatibility
- phosphate-free

Application areas

Neutralizer for automated treatment of dental instruments after alkaline cleaning (thermodent® powder, thermodent® alka clean), prevents coverings and to optimize cleaning results. Suitable for all common washers/disinfectors (WD).

Instructions for use

Dosage: approx. 0,1 - 0,2 % (1 - 2 ml/l).
Dosing with machine integrated dosing pumps.



thermodent[®] neutralizer

Product data

Components:
citric acid.

Chemical-physical data

| | |
|--------------------|------------------------------------|
| Color | colourless |
| Density | ca. 1,17 g/cm ³ / 20 °C |
| Flash point | Not applicable |
| Form | liquid |
| pH | ca. 2 / 20 °C |
| Viscosity, dynamic | ca. 4 mPa*s / Method : ISO 3219 |

Information for order

| Item | Delivery form | Item no. |
|-------------------------------|---------------|------------|
| thermodent neutralizer 5 l KA | 1/Canister | on request |

These products are not available in every country. For more information please contact our local subsidiary or distributor.

Application aids

| Application aids | Item no. |
|----------------------|----------|
| Can key for 5 + 10 l | 135810 |

Related Products

- thermodent[®] alka clean
- thermodent[®] clear

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: www.schuelke.com. For individual questions: Customer Sales Service
Phone: +49 40 52100-666
E-Mail: info@schuelke.com



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 40 - 52100 - 0
Fax +49 40 - 52100 - 318
www.schuelke.com
info@schuelke.com