

schülke -†



Mildly alkaline high-performance cleaner for automated reprocessing of flexible endoscopes in AER

thermosept® EndoCleaner

Our Plus

Application areas

thermosept® EndoCleaner is a modern, mild alkaline cleaner especially for the effective treatment of flexible endoscopes and other endoscopic instruments in washer disinfectors. The efficient cleaning performance of thermosept® EndoCleaner is used to optimally prepare the devices for the process step of chemothermal disinfection. The powerful cleaning performance and good material compatibility of thermosept® EndoCleaner could be confirmed in extensive practical tests. thermosept® EndoCleaner is suitable for use in AER from leading manufacturers, such as Belimed, BHT, Getinge, Olympus, Steelco, Steris and Wassenburg.

Instructions for use

Standard concentration:

0.5 % (5 ml/l); 5 min.; 45 °C

Dosage and cleaning time depending on the level of soiling:

0.4 % - 0.6 % (4 - 6 ml/l); 3 - 10 min.; 35 °C - 55 °C.

Preferably use soft or deionised water. Please follow the application instructions of the machine manufacturers!

thermosept® EndoCleaner is compatible with the disinfection components thermosept® ED (glutaraldehyde based) and thermosept® PAA (peracetic acid based) in the chemothermal treatment process for flexible endoscopes. Please follow the instructions of use of the disinfectants thermosept® ED or thermosept® PAA.



thermosept® EndoCleaner

Product data

Composition:

Labelling according to Regulation (EC) No. 648/2004: 5 - 15 % anionic surfactants, enzymes, < 5 % nonionic surfactants, < 5 % polycarboxylate, Other ingredients: corrosion inhibitors, solubiliser.

Chemical-physical data

Color	yellow
Density	ca. 1,10 g/cm ³ / 20 °C / 1.013 hPa
Flash point	> 100 °C / Method : DIN 51755 Part 1
Form	liquid
pH	ca. 11 / 20 °C
Viscosity, dynamic	ca. 9 mPa*s / Method : ISO 3219

Special advice

Keep container tightly closed. Protect from frost, heat and sunlight. Store at room temperature in the original container (Storage temperature: 5 - 25 °C).

Information for order

Item	Delivery form	Item no.
thermosept EndoCleaner Drumt. 200 l FA	1/drum(s)	on request
thermosept EndoCleaner 5 l KA	1/Canister	on request
thermosept EndoCleaner BHT Pin 5 l KA	1/Canister	on request
thermosept EndoCleaner 20 l KA	1/Canister	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

Related Products

- thermosept® ED
- thermosept® PAA additive
- thermosept® PAA base

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: <http://www.schuelke.com/>.
For individual questions: Customer Sales Service Phone: +49 40 52100-666 E-Mail: info@schuelke.com



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 40 - 52100 - 0
Fax +49 40 - 52100 - 318
www.schuelke.com
info@schuelke.com