

## schülke wet wipes dispenser

### Filling under hygienic conditions

#### Step 1

We recommend that you disinfect your hands and the work surface and wear an overall, protective goggles and gloves\*. This will protect you and the schülke wipes.

\* as required by paragraph 4.1.3 of TRBA 250



#### Step 2

Open the lid of the wet wipe dispenser and insert the unpacked fleece roll into the dispenser. Pull the inmost wipe a little from the centre.



#### Step 3

Pour 2 litres of ready-to-use product\*\* or 3 litres of diluted concentrate solution once slowly in a spiral shape over the roll of wipes in the bucket. (schülke wipes 40 / 70 = 1,5 litre).

\*\* antifecht® N liquid, buraton® rapid, mikroizid® AF liquid, mikroizid® universal liquid, mikroizid® sensitive liquid, perform® advanced alcohol EP, perform® classic alcohol EP, terralin® liquid



#### Step 4

Now pull the inmost wipe of the roll into the star-shaped dispensing opening of the lid and dispose of it.



#### Step 5

Place the lid onto the dispenser and close it until it locks into place.

Now carefully close the protective cap of the dispenser.



#### Step 6

The wipes take 30 minutes to become fully saturated.



Wait  
30 minutes



#### Step 7

In the meantime, complete each charge label supplied with each cloth roll and stick it onto the field provided.



#### Step 8

The schülke wet wipes dispensing system is now ready for use. All schülke products have a shelf life of 28 days. Except for terralin® PAA with 12 hours of shelf life.



### Hygienic refilling

1. Please dispose any remaining product and wipes after the expiry date or when refilling.
2. **Clean** the dispenser and the lid thoroughly with an efficient and absorbent wipe (e.g. mikroizid® sensitive wipes), to remove any soiling.
3. **Disinfect** the lid and the dispenser thoroughly. We recommend using alcoholic disinfectant wipes like the mikroizid® AF wipes.
4. The lid and the dispenser can also be reprocessed for 10 minutes at 70°C or for 1 minute at 90°C.
5. The dispenser and the lid must dry after disinfecting.  
**The refilling of the dried dispenser must occur in the same way as the initial filling.**



Discard solution



Dispose of wipes



Clean, disinfect & dry



Clean & disinfect