



Alcohol free, dual action cleaner and disinfectant for alcohol sensitive surfaces

Effective against enveloped viruses

## mikrozid<sup>®</sup> alcohol free liquid

### Our Plus

- alcohol free, making it ideal for use on all surfaces including dental chairs and alcohol sensitive materials
- added surfactants for boosted cleaning performance
- cleaning and disinfection, two complete actions in one product
- effective against bacteria, yeast and enveloped viruses
- rapid 1 minute contact time, making it ideal for use between patients
- ready to use solution

### Application areas

For the disinfection of medical devices and other environmental surfaces. Especially suitable for surfaces sensitive to alcohol (e.g. leather, PVC and acrylic glass).

### Instructions for use

mikrozid<sup>®</sup> alcohol free liquid is a ready-to-use solution. Apply mikrozid<sup>®</sup> alcohol free liquid undiluted to objects and surfaces and allow to take effect. Ensure that surfaces are thoroughly wetted. If necessary, remove excess product with a dye-free disposable wipe. You can apply the ready-to-use solution either as a spray or as a foam disinfectant.

### Microbiological efficacy

Efficacy	Concentration	Contact time
bactericidal in accordance with VAH Guidelines with short contact times	ready-to-use	1 min.
MRSA	ready-to-use	1 min.
virucidal against enveloped viruses	ready-to-use	1 min.
Bovine viral diarrhea virus	ready-to-use	1 min.

Efficacy	Concentration	Contact time
Vaccinia virus	ready-to-use	1 min.
Rotavirus	ready-to-use	1 min.

CE 0297



# mikrozid<sup>®</sup> alcohol free liquid

## Product data

Composition: 100 g solution contains the following active substances: 0.26 g Quaternary ammonium compounds, benzyl-C12-C18-alkyldimethyl, chlorides, 0.26 g Didecyldimethylammonium chloride, 0.26 g Quaternary ammonium compounds, C12-14-alkyl [(ethylphenyl) methyl] dimethylammonium, chlorides.

## Special advice

**Use biocides safely. Always read the label and product information before use.**

For all of the related safety information please refer to the Safety Data Sheet. To view the Safety Data Sheet please visit [www.schuelke.com](http://www.schuelke.com)

Wearing gloves (e.g. of nitrile rubber) is recommended. Keep container tightly closed. Protect from frost, heat and sunlight. Store at room temperature in the original container. Only recycle completely emptied packaging. Leftover product can be disposed of in accordance with EWC 070601. Do not use near ignition sources. Do not exceed usage of more than 50ml per m<sup>2</sup>. Avoid contact with the eyes. Store out of the reach of children. Not for final disinfection of semicritical and critical medical devices.

## Information for order

Item	Delivery form	Item no.
mikrozid alc. free liquid -GB- 1 I FL	10/Carton	165720
mikrozid alc. free liquid -GB- 5 I KA	1/Canister	165721

## Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.

## Expert opinion and information

Please visit our website for an overview of all available literature/reports on all schülke products online at <http://www.schuelke.com/>

For any queries you might have about the product or using it please call 0114 2543500 or E-mail [mail.uk@schuelke.com](mailto:mail.uk@schuelke.com)

Item no.	GTIN no.
165720	4032651657209
165721	4032651657216



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

**schülke UK**  
Schülke & Mayr UK Ltd  
Cygnets House, Meadowhall  
Sheffield, S9 1AT  
Phone +44-114-254 35 00  
Fax +44-114-254 35 01  
[mail.uk@schuelke.com](mailto:mail.uk@schuelke.com)

**schülke Headquarters**  
Schülke & Mayr GmbH  
Robert-Koch-Str. 2  
22851 Norderstedt, Germany  
Phone +49 40 - 52100 - 0  
Fax +49 40 - 52100 - 318  
[www.schuelke.com](http://www.schuelke.com)  
[info@schuelke.com](mailto:info@schuelke.com)