



Alcohol free, dual action cleaner and disinfectant for alcohol sensitive surfaces.

1 minute contact time  
(EN16615, 4-field test)

## mikrozid® alcohol free wipes

### Our Plus

- ideal for all surfaces including dental chairs and alcohol sensitive materials
- added surfactants for boosted cleaning performance
- cleaning and disinfection, two complete actions with one product
- effective against bacteria, yeast and viruses, including enveloped viruses
- rapid one minute contact time (EN16615 4-field test), making it ideal for use between patients

### Application areas

For the disinfection of medical devices and other environmental surfaces. Especially suitable for surfaces sensitive to alcohol (e.g. leather, PVC and acrylic glass)

### Instructions for use

Wipe surfaces thoroughly with the impregnated wipe, ensure complete wetting and allow to take effect. Make sure that all visible soiling is removed before disinfection and the lid is tightly closed after use. If necessary, use several wipes for large areas.

#### Refilling

First disinfect hands and wear new protective gloves. During the refilling process, please ensure that the gloves are not contaminated by other activities. On every refill, subject the lid closure to disinfectant cleaning with an alcohol based surface disinfectant.

### Microbiological efficacy

Efficacy	Concentration	Contact time
bactericidal <i>in accordance with VAH Guidelines with short contact times</i>	ready-to-use	1 min.
MRSA	ready-to-use	1 min.

Efficacy	Concentration	Contact time
yeastocidal <i>in accordance with VAH Guidelines with short contact times</i>	ready-to-use	1 min.
virucidal against enveloped viruses	ready-to-use	1 min.
Bovine viral diarrhea virus	ready-to-use	1 min.
Vaccinia virus	ready-to-use	1 min.
Rotavirus	ready-to-use	1 min.

CE 0297



# mikrozid<sup>®</sup> alcohol free wipes

## Product data

### Composition:

100 g solution contains the following active substances:  
0.26 g Quaternary ammonium compounds, benzyl-C12-C18-alkyldimethyl, chlorides, 0.26 g Didecyldimethylammonium chloride, 0.26 g Quaternary ammonium compounds, C12-14-alkyl [(ethylphenyl)methyl] dimethylammonium, chlorides.  
One dispenser or refill contains 200 ready-to-use Jumbo wipes.

## Special advice

### Use biocides safely. Always read the label and product information before use.

For all of the related safety information please refer to the Safety Data Sheet. To view the Safety Data Sheet please visit [www.schuelke.com](http://www.schuelke.com)

Wearing gloves (e.g. of nitrile rubber) is recommended. Keep container tightly closed. Protect from frost, heat and sunlight. Store at room temperature in the original container. Only recycle completely emptied packaging. Leftover product can be disposed of with household waste. Do not use near ignition sources. Avoid contact with the eyes. Store out of the reach of children. Not for final disinfection of semicritical and critical medical devices.

## Information for order

Item	Delivery form	Item no.
mikrozid al.free wip Ref -GB- 200 ST BT	12/Carton	70001449
mikrozid alc. free wipes -GB- 200 ST DS	10/Carton	165723

## Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.

## Expert opinion and information

Please visit our website for an overview of all available literature/reports on all schulke products online at <http://www.schuelke.com/>

For any queries you might have about the product or using it please call 0114 2543500 or E-mail [mail.uk@schuelke.com](mailto:mail.uk@schuelke.com)

Item no.	GTIN no.
165722	4032651657223
165723	4032651657230
70001449	4032651092147



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

**schülke UK**  
Schülke & Mayr UK Ltd  
Cygnets House, Meadowhall  
Sheffield, S9 1AT  
Phone +44-114-254 35 00  
Fax +44-114-254 35 01  
[mail.uk@schuelke.com](mailto:mail.uk@schuelke.com)

**schülke Headquarters**  
Schülke & Mayr GmbH  
Robert-Koch-Str. 2  
22851 Norderstedt, Germany  
Phone +49 40 - 52100 - 0  
Fax +49 40 - 52100 - 318  
[www.schuelke.com](http://www.schuelke.com)  
[info@schuelke.com](mailto:info@schuelke.com)