

# schülke -†



Mildly alkaline high-performance cleaner for automated reprocessing of flexible endoscopes in AER / EWD

## thermosept® EndoCleaner

### Our Plus

- outstanding cleaning performance through synergistic active substance combination of enzymes and surfactants
- optimised material compatibility; suitable for endoscopes and AER of all leading manufacturers
- highly economical due to low application concentration
- low-foaming formulation

### Application areas

thermosept® EndoCleaner is a modern, mild alkaline cleaner especially for the effective treatment of flexible endoscopes and other endoscopic instruments in washer disinfectors. The efficient cleaning performance of thermosept® EndoCleaner is used to optimally prepare the devices for chemothermal disinfection. The powerful cleaning performance and material compatibility of thermosept® EndoCleaner has been confirmed in extensive practical tests. thermosept® EndoCleaner is suitable for use in AER / EWD from leading manufacturers, such as Belimed, BHT, Getinge, Olympus, Steelco, Steris and Wassenburg.

### Instructions for use

Standard concentration:

0.5 % (5 ml/l); 5 min.; 45 °C

Dosage and cleaning time depending on the level of soiling:

0.4 % - 0.6 % (4 - 6 ml/l); 3 - 10 min.; 35 °C - 55 °C.

Preferably use soft or deionised water. Please follow the application instructions of the machine manufacturers.

thermosept® EndoCleaner is compatible with the disinfection components and thermosept® PAA (peracetic acid based) in the chemothermal treatment process for flexible endoscopes.



# thermosept<sup>®</sup> EndoCleaner

## Product data

### Composition:

Labelling according to Regulation (EC) No. 648/2004: 5 - 15 % anionic surfactants, enzymes, < 5 % nonionic surfactants, < 5 % polycarboxylate, Other ingredients: corrosion inhibitors, solubiliser.

## Special advice

Keep container tightly closed. Protect from frost, heat and sunlight. Store at room temperature in the original container (Storage temperature: 5 - 25 °C).

For all of the related safety information please refer to the Safety Data Sheet. To view the Safety Data Sheet please visit [www.schuelke.com](http://www.schuelke.com)

## Information for order

Item	Delivery form	Item no.
thermosept EndoCleaner 5 l KA	1/Canister	127610

## Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

## Expert opinion and information

Please visit our website for an overview of all available literature/reports on all schulke products online at <http://www.schuelke.com/>

For any queries you might have about the product or using it please call 0114 2543500 or E-mail [mail.uk@schuelke.com](mailto:mail.uk@schuelke.com)



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

**schülke UK**  
Schülke & Mayr UK Ltd  
Cygnet House, Meadowhall  
Sheffield, S9 1AT  
Phone +44-114-254 35 00  
Fax +44-114-254 35 01  
[mail.uk@schuelke.com](mailto:mail.uk@schuelke.com)

**schülke Headquarters**  
Schülke & Mayr GmbH  
Robert-Koch-Str. 2  
22851 Norderstedt, Germany  
Phone +49 40 - 52100 - 0  
Fax +49 40 - 52100 - 318  
[www.schuelke.com](http://www.schuelke.com)  
[info@schuelke.com](mailto:info@schuelke.com)